
Instructions

Product Name: Half Face Mask with Particle Filter

Base material: Spunbond, Meltblown, Nano Line

Specifications: 160 mm * 172 mm

Type: Non-medical mask FFP2 NR

Model: BR1MSK-001

Standard: EN149: 2001 + A1: 2009

Production date: Month / Year

Storage Life: 3 years. Consult the manufacturer's information before use. (see package for production information)

Manufacturer: BRBEN TEKSTIL SAN VE TIC AS

Address: Başıpınar OSB district 2. OSB 83207 street No:2 Şehitkamil/Gaziantep, Turkey

Phone: +90 536 777 60 99

Area of Use

Half face mask with suitable filter to prevent all kinds of particles.

Not suitable for respiratory protection against harmful gases and vapors.

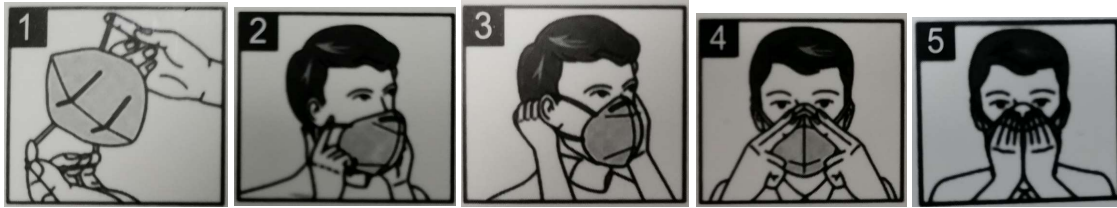
It is not suitable for use in hypoxia environments, underwater work and with fire areas.

Storage and Transportation: During storage and transportation, the product should be sealed, not damaged and stained and must be protected from moisture. Avoid direct sunlight.




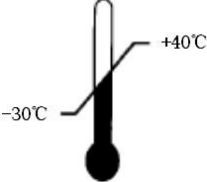

Storage method: Please store in a dry and ventilated environment, storage temperature -30 ~ 40 °C, and relative humidity should not exceed 80%.

Application

1. Look at the mask, nose clip should be on top.
2. Hold the mask against your face and chin.
3. Pull the straps on both sides and hang them on the ears and adjust until you feel comfortable.
4. Put your fingers in the middle of the metal nose clip and move your fingers to either side until the nose clip is fully pressed into the nose bend shape.
5. Put your hands on your face to check the firmness of your face.



Logos and Meanings

	<p>Disposable</p>		<p>Keep away from direct sunlight</p>
	<p>Storage conditions maximum relative humidity</p>		<p>Temperature range of storage conditions</p>
	<p>See information provided by the manufacturer</p>	<p>CE 1463</p>	<p>CE mark and Notified Institution Number</p>

Warnings

1. Before using the masks, check that they are suitable for the intended use. User, must be adequately trained before use and make sure that the masks fit properly.
2. The mask is disposable and cannot be reused.
3. Using masks with facial hair will cause leakage problem in case the facial hair is not covered.
4. If it is damaged or if breathing resistance increases, discard and replace the mask.
5. Do not use the mask in explosive gases, heavily polluted environments, oxygen-free environment, fire environment and underwater work.

6. Remove patients with heart disease or other illness when uncomfortable.
7. Masks marked "NR" will not be used for more than one shift.
8. Do not use this mask when the oxygen concentration is below 19.5%.
9. Do not use in explosive atmospheres.
10. If the package is damaged, it is strictly forbidden to use it.